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
*No. 1 choice
of hospitals
& mothers*

- EN** Instructions for use
- RU** Инструкция по применению
- HU** Használati utasítás
- CS** Návod k použití
- TR** Kullanma talimatları
- PL** Instrukcja obsługi

EN Parts List XIII
RU Список деталей XIII
HU Alkatrészek listája XIII
CS Seznam dílů XIII
TR Parça Listesi XIII
PL Wykaz części zamiennych XIII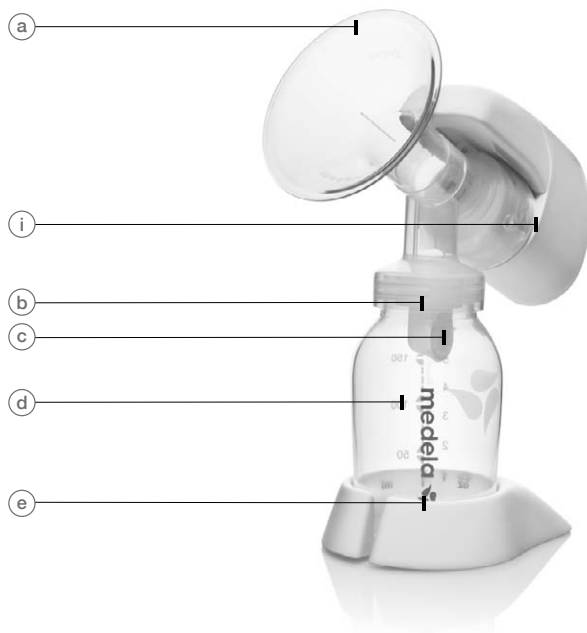

EN	Instructions for use	4
RU	Инструкция по применению	13
HU	Használati utasítás	22
CS	Návod k použití	31
TR	Kullanma talimatları	40
PL	Instrukcja obsługi	49
	EMC	58

I Mini Electric™ Breastpump

Mother's milk is the best nutrition for babies. Mini Electric gives you and your baby optimal support during the breastfeeding period. The attractive small electric breastpump is especially suitable for occasional pumping at home or on the go.

II Indications For Use

The Mini Electric Breastpump is a powered breastpump:

- To be used by lactating women to express and collect milk from their breast during absences from the baby (e.g. an evening out from baby, working part-time).
- To relieve engorgement.
- For home use and on the go.

III Important Safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

Read all instructions before use.

- *Use the product only for its intended use as described in this instruction manual.*
- *Only use the mains adapter that comes with the Mini Electric Breastpump.*
- *Make sure the voltage of the mains adapter is compatible with the power source.*
- *Plug the mains adapter into Mini Electric Breastpump first and then into the wall socket.*
- *Always unplug electrical devices immediately after using.*
- *The product should never be left unattended when plugged into a power source.*
- *Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.*
- *Keep the mains adapter away from heated surfaces.*
- *Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.*
- *Do not use outdoors.*
- *Keep appliance dry and do not place in direct sunlight.*
- *Do not get the product wet.*
- *Do not use while bathing or showering.*

- *Do not place in or drop motor unit into water or other liquid.*
- *Do not store or use this product where it can fall or be pulled into a tub or sink.*
- *Do not spray or pour liquid directly onto the pump.*
- *Do not reach for any electrical device if it has fallen into water. Unplug immediately.*
- **Use Medela original accessories only.**
- *Check pump kit components for wear or damage.*
- *This is a single user product. Use by more than one person may present a health risk and voids the warranty.*
- *Pump only with the motor in an upright position.*
- *Do not use while sleeping or overly drowsy.*
- *Supervision is necessary when this product is used in the vicinity of children.*
- *If problems or pain occur, consult your lactation consultant or physician.*
- *Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.*
- *Portable and mobile radio frequency communications equipment can affect the Mini Electric Breastpump.*
- *The Mini Electric Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.*
- *There are no user-serviceable parts inside the Mini Electric. Do not attempt to repair the Mini Electric Breastpump yourself.*

SAVE THIS INSTRUCTION MANUAL

IV Assembly Instructions

1

b + c

Snap membrane onto valve until membrane lies completely flat.

2

a1 a2

+

b c

Push assembled valve and membrane onto breastshield. Ensure sidewise position of valve as indicated on picture.

3

a b c

+

d

Screw bottle into breastshield.

4

a b c d

+

i

Align the raised edge on the breastshield back with the ridge on the underside of the battery cover and press the motor firmly onto the shield. Lock the motor onto the breastshield by rotating clockwise until the stop.


V Operation

Electrical operation

Insert the metal plug on the end of the mains adapter cord into the hole on the side of the motor unit.
Plug the electrical connector on the end of mains adapter cord into power jack (3 V DC).


Battery operation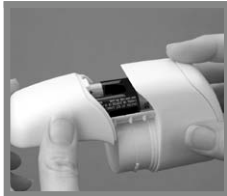

To install the batteries, slide off the cover of the battery compartment.
Insert the batteries in the correct position as indicated inside the battery compartment and slide on the cover back into place (AA Mignon LR6).
The batteries do not need to be removed if the mains adapter is used for pumping.

VI Pumping

Be comfortably seated and relax. Use a footrest, when pumping or when feeding your baby. Elevating your feet puts you into proper sitting position to avoid stress on the lower back and perineum.

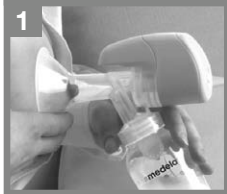Center your breastshield over your nipple.

Do not tilt or overfill bottles when pumping.

VI Pumping

2

Set the vacuum regulator to minimum and switch on the breastpump by pushing the slide switch upwards.

Adjust the vacuum to your **Maximum Comfort Vacuum™**. This is the highest vacuum you can use and still feel comfortable (different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

Research has shown that you will pump more efficiently – meaning get more milk in less time – when pumping at Maximum Comfort Vacuum™.

Once finished pumping, switch off the breastpump, unplug it, and disassemble the pump kit items. Close the bottle and store it for later use.

Use bottle stand to prevent tip over.

VII Breastshield Fitting

Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

Medela PersonalFit™ Breastshield: Available in five sizes – because one size does not fit all.

VIII Cleaning and Hygiene

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Disassemble all parts prior to cleaning. Especially be sure to remove the yellow valve head from the breastshield and separate the white valve membrane from the valve head.

Prior to first use and once daily

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+f+g+h), except motor unit.
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

Alternatively, the disassembled parts can be sanitized in top rack of dishwasher or steam-sterilized.

Note: Food colorings may discolor components which will not affect performance.

After each use

- Disassemble and handwash all parts of your pump kit that come in contact with the breast and milk (a+b+c+d+f+g+h) in warm, soapy water (except motor unit).
- Rinse in clear water.
- Dry parts with a clean towel.
- Use a damp (not wet) cloth to wipe the motor unit after each use.

Hints

After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

VIII Cleaning and Hygiene

Hygiene

- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Avoid touching the inside of bottles and lids.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol which can dry the skin and cause cracked nipples.

IX Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- *When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.*
- *Label bottles/bags with the date of pumping.*
- *Do not store breastmilk in the door of the refrigerator.*
- *In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).*
- *Gently shake the bottle/bag to blend any fat that has separated.*
- *Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).*

X Troubleshooting

If the motor is not running

Check

- mains/battery power is present
- the device is switched on

If there is low or no suction

Make sure

- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- the white valve membrane is laying flat against the yellow valve head. Check that the valve head and membrane are clean and not damaged. See «Assembly Instructions».
- the motor unit is securely locked onto the breastshield.

If there is an overflow

If milk accidentally gets into the pump

- Turn off the pump and unplug from power source.
- Remove motor unit from breastshield.
- Wipe off any breastmilk on motor unit and breastshield and let the motor unit pump air for at least 30 seconds.
- If necessary, clean the pump kit according to cleaning instructions «Prior to first use and once daily».

XI Warranty

This is a single user product. Use by more than one person may present a health risk and voids the warranty. 6 months warranty for the unit from the date of purchase, excluding pump kit. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons.

XII Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects in the environment and on public health.

XIII Parts List

Article numbers

a1	Breastshield (24 mm)	810.7084
a2	Connector	810.7088
b	Valve head	810.0475
c	Valve membrane	820.7013
d	Bottle (150 ml)	811.0021
e	Bottle stand	810.0462
f	Cap	810.0474
g	Lid	810.7062
h	Disc	810.0473
i	Motor unit	600.0579
j	Mains adapter (EURO)	920.0015
	Mains adapter (UK)	920.0017
	Mains adapter (US/JP)	920.0013
	Mains adapter (AUS/NZ)	920.0016

*Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.
Subject to change.
Content may vary from picture.*

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
Electromagnetic Compatibility (EMC, IEC 60601-1-2:2001)

IEC 60601-1-2, Table 201







Electromagnetic Emissions		
The Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the Mini Electric should assure that it is used in such environment.		
Emission Tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Mini Electric uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Mini Electric is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

Warning: The Mini Electric should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Mini Electric should be observed to verify normal operation in the configuration in which it will be used. The Mini Electric was not tested for immunity to electromagnetic disturbances.

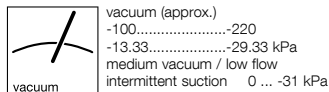EN Technical specifications**RU** Технические спецификации**HU** Technikai adatok**CS** Technické specifikace**TR** Teknik Özellikler**PL** Specyfikacja techniczna

	IN	OUT
50 Hz	Euro 230 V UK	3V <u>=====</u> 0.5 A
	Aus 240 V NZ	
60 Hz	US 120 V JP	

===== Direct current
 ~~~~~ Alternating current

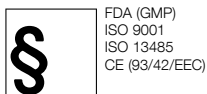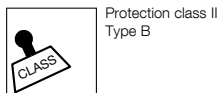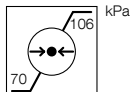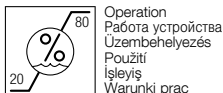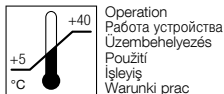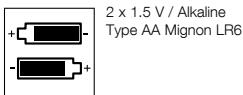CE 0123

Please see accompanying papers  
 Пожалуйста, прочтите прилагающуюся документацию  
 Figyelem, be kell tartani a mellékelt dokumentumokban leírtakat  
 Pozor, informovat se v průvodní dokumentaci  
 Lütfen ilıřketeki belgelere bakın  
 Uwaga, postępować zgodnie z dokumentacją

Must not be disposed of together with household refuse  
 Не утилизируйте вместе с бытовыми отходами  
 Nem szabad a háztartási hulladékokkal együtt ártalmatlanítani  
 Nesmí se odstraňovat zároveň s domácím odpadem  
 Ev atıkları için kullanılan çöpe atılmamalı  
 Nie wolno poddawać utylizacji razem z odpadami domowymi

MINI ELECTRIC BREASTPUMP  
 WITH RESPECT TO ELECTRIC SHOCK FIRE,  
 AND MECHANICAL HAZARDS ONLY IN  
 ACCORDANCE WITH  
 UL2801-1/CAN/CSA C22.2 No. 601.1 22PL

[www.medela.com](http://www.medela.com)

Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
[www.medela.com](http://www.medela.com)

#### **International Sales**

Medela AG  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 (0)41 769 51 51  
Fax +41 (0)41 769 51 00  
[ism@medela.ch](mailto:ism@medela.ch)  
[www.medela.com](http://www.medela.com)

#### **Poland**

Medela Poland Sp. z o.o.  
Ul. Lewinowska 8  
03-684 Warszawa  
Phone +48 22 864 38 65  
Fax +48 22 865 12 50  
[biuro@medela.pl](mailto:biuro@medela.pl)  
[www.medela.pl](http://www.medela.pl)

#### **Russia**

OOO Medela  
Vavilova str. 97, entrance 3  
117335 Moscow / Russia  
Phone +7 (495) 980 6194  
Fax +7 (495) 980 6194  
[info@medela-russia.ru](mailto:info@medela-russia.ru)  
[www.medela-russia.ru](http://www.medela-russia.ru)